# USER INSTRUCTIONS

Model name (Classification):	FFP1 NR	FFP2 NR	FFP3 NR
ZH3001.ZH3001V, ZH4001, ZH4001V	√	x	x
ZH3161,ZH3161V	√	√	x
ZH3310, ZH3310V, ZH4310/ZH4310V	√	√	x
ZH3071.ZH3071V	√	√	x
ZH3361, ZH3361V, ZH4361, ZH4361V	√	√	√
ZH4071.ZH4071V	√	√	√
ZH3081.ZH3081V, ZH4081, ZH4081V	√	√	√
ZH3291V	√	√	√

"NR" if the particle filtering half mask is limited to single shift use only.

**International Certification** : ISO9001 : 2015, ISO14001 : 2015, ISO45001: 2018


These products conform to the European Norm EN149 : 2001 + A1:2009 and Regulation (EU) 2016/425.

Ways for storage: keep in damp proof, moisture proof, with no rain-drenching, and ventilated room.

**Guarantee Period**: 3 Years

**Manufacture Date and lot No.:** Check the outside of the inner box

## Fitting Instructions:

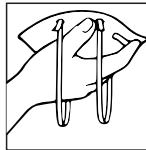1. Hold the mask in hand with the nosepiece up. If folded mask, then open the folded mask with the nosepiece up and hold the mask in hand, allow headbands to hang freely.


2. Position the mask under the chin covering mouth and nose.

3. Pull the lower headband over the head and position below ears. Pull the top headband on the back of head above ears for best fit.

4. Press soft nosepiece to conform snugly around the nose.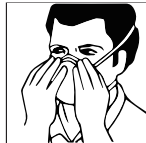

5. To check fit, cup both hands over the mask and exhale vigorously. If air flows around nose, tighten the nosepiece. If air leaks around the edge, reposition and adjust the headband for better fit.  
Retest the seal and repeat the procedure until the mask is sealed properly.

## Applications:

- FFP1 NR : Protection against non-toxic solid and liquid aerosols(e.g. oil-mists) in concentrations up to 4xTLV(Threshold Limit Value).
- FFP2 NR : Protection against non-toxic solid and liquid aerosols(e.g. oil-mists) in concentrations up to 12xTLV(Threshold Limit Value).
- FFP3 NR : Protection against non-toxic solid and liquid aerosols(e.g. oil-mists) in concentrations up to 50xTLV(Threshold Limit Value).

Respiratory protection is only effective if it is correctly selected, fitted and worn throughout the time when the wearer is exposed to hazards.

Functions: Suited to mining, building construction, grinding, pharmaceutical, casting, agriculture, farming, electronic, food processing, cement, textile, carving, heavy metal pollution, gardening, glass...etc workshops.

## Warning:

1. Please check the visible damage, dirt and the expiry date before use.
2. Failure to follow all instructions and limitations on the use of this product could reduce the effectiveness of respirator and result in sickness or death.
3. A properly selected respirator is essential to protect your health. Before using this respirator, consult an industrial hygienist or occupational safety professional to determine the suitability for your intended use. Always check the respirator for fit prior to use.
4. This product does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. Do not use this respirator when oxygen concentration is less than 19.5%.
5. Do not use when concentrations of contaminants are immediately dangerous to health or life. Do not use this respirator in explosive environment.
6. Leave work area immediately and return to fresh air if, a) breathing becomes difficult, or b) dizziness or other distress occurs;
7. Facial hairs or beards and certain facial characteristics may reduce the effectiveness of this respirator.
8. Never alter or modify this respirator in any way.
9. Do not use for more than one shift (8 hours). No maintenance necessary. Discard the used respirator after single shift use.
10. Keep respirator in the packaging box away from direct sunlight until use.

**Manufacturer:** Shanghai Zhongzhi Health Articles Co.,Ltd.

Address: No. 1068, Mingye Road, Sheshan Industrial Zone, Songjiang District, Shanghai

Tel: 021-57792881

Fax: 021-57797077

E-Mail: support@zhongzhimask.com

Website: <http://www.zhongzhimask.com>

## Notified Body:

BSI Group The Netherlands B.V. (2797)

Address: Say Building, John M. Keynesplein 9, 1066 EP, Amsterdam, Netherlands

## EU Authorized Representative:

Lotus NL B.V. Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague, Netherlands

Shelf life: 3 years

Temperature range of storage conditions: -20°C to +40°C

Maximum relative humidity of storage condition: <30%

Batch code

See information supplied by the manufacturer

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**Hersteller:** Shanghai Zhongzhi Health Articles Co., Ltd.
**EU-Repräsentant:** Lotus NL B.V. Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague, Netherlands